


Insert cable lead and switch on at back of machine.

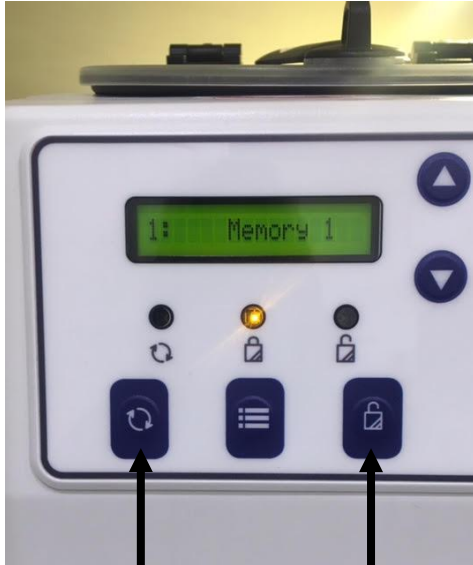

START

OPEN LID

(or for emergency stop)

Screen view after switching on

The centrifuge is already set using Memory 1 (RCF of 1503g, RPM of 3500 for 10min)

Depress OPEN LID button, open lid and insert white bucket fillers for 11ml tubes

After placing a balancing tube filled with water*, diagonally opposite to the filled Cellenis PRP tube, push the START button to begin centrifuging

- The lid has a safety locking mechanism
- Lid remains open for 15 seconds after the lid OPEN button is depressed
- Seconds countdown is shown on screen. If the lid is not opened in time, depress OPEN button again
- After 10min centrifuging is complete, three beeps are sounded
- The lid can then be opened for 50 seconds
- Seconds countdown is shown on screen. If the lid is not opened in time, depress OPEN button again
- The lid open button is ready when the red light above is on.

Trouble Shooting

- *If the yellow light above is not on, check that the lid open and close latch is correctly in place.*
- *If various buttons have been inadvertently depressed because the yellow or red light didn't come on, and now nothing seems to be working..... take a big breath, then simply depress the menu button (the button in the middle with horizontal lines) and scroll to Memory 1. It will then restore the programme to the original setting for Cellenis PRP.*