
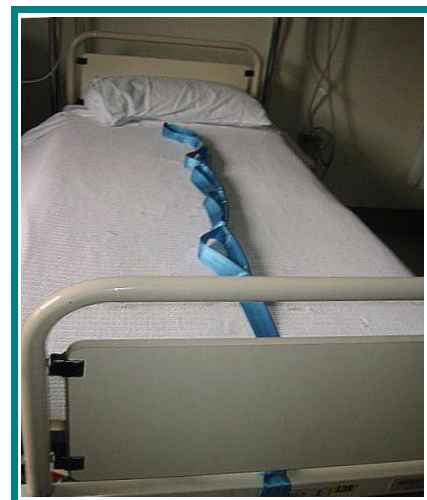


BED LADDER STRAP USER GUIDE

Order Code: BLS 08

The Patient Handling Bed Ladder Strap is made from strong quality webbing with sewn loop handholds down its length. It allows clients to pull themselves upright in bed by gripping each handhold loop in turn. If the patient cannot grip, they can place their wrist or forearm through each loop to pull themselves up. As well as assisting with sitting up, the strap can be used to steady movement when standing or moving from bed to chair.

- Provides exercise for the upper body during use.
- 1.9 metres in length. Five handholds and bed attachment loop.
- Strong webbing construction suitable for heavier clients. Breakload 250 Kg.
- Easily fitted to most types of bed. The largest end loop can be either looped through bottom bed rail and back through itself or around a bed leg. The strap then lies on the bed without getting in the way, ready for use.
- CE Class 1 Medical Device
- Hand or machine washable - colour blue.

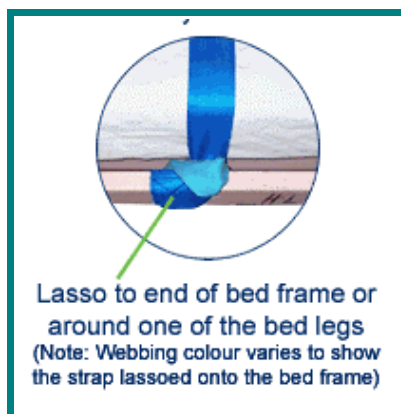

THESE ARE GENERAL GUIDELINES ONLY. PLEASE SEEK ACCREDITED TRAINING.

Patient Handling UK, 2 East Court, Burley, Oakham, Rutland LE15 0BD

Freephone: 0800 083 6071 Fax: 0208 114 9266

Email: contact@patienthandling.org.uk
Web: www.patienthandling.org.uk