

STOCKING SLIDER

Order Code STS

Open Toe Compression stockings help to prevent pressure sores.
The Patient Handling Stocking Slider helps to fit the stockings over the foot and heel.
The Slider can also be used to help with tight fitting sleeves when fitted over the hand.

- 1) Tuck the Stocking Slider down inside itself to make a pouch for the foot.
- 2) Put the Stocking Slider over the foot and ankle.
- 3) Gather up the Compression Stocking, fit it over the Stocking Slider and work up the leg into place.
- 4) After the stocking is fully on the foot and leg, pull the Stocking Slider end loop away from the foot.
- 5) Keep pulling the Stocking Slider until it is completely free.
- 6) Tuck the Stocking Slider back inside itself ready to use again.

Washing: Hand or machine wash only, up to 80°C.
Do not tumble dry or wring - air dry only.
Colour: Red



Patient Handling UK, 2 East Court, Burley, Oakham, Rutland LE15 7FH

Freephone: 0800 083 6071 Email: contact@patienthandling.org.uk
Web: www.patienthandling.org.uk