



## TRANSFER TURNTABLE

- The Transfer Turntable helps when transferring patients from one seated position to another. The smooth, controlled rotation avoids jerks and twists which might damage a patient's limbs or joints. It provides greater control for the carer who can adopt a safer working position reducing the strain and risk to their back.
- The Transfer Turntable should only be used on a flat, firm, surface. The Carer should always assist the patient when they are using it. The Turntable should be large enough to accommodate the patient's feet.
- Turntable Diameter: 41 cm. (16 inches ) Height off ground: 2 cm. ( $\frac{3}{4}$  inch) Weight: 1 Kg.
- The Patient's feet should be placed in the centre of the turntable and the Patient assisted into an upright standing position. They can then be rotated, in either direction, round to the new position and assisted into the new seating position. A Walk Belt could be used in conjunction with the turntable to aid the standing and sitting procedures.
- Maximum User Weight for the Turntable is 125 Kg. (19  $\frac{1}{2}$  Stone)
- The Turntable can be gently cleaned with a non-abrasive detergent cleaning agent, rinsed and dried. No solvents or chlorine cleaners should be used nor should it be autoclaved or irradiated.
- CAUTION Approved assessments and handling techniques must be used with all moving and handling procedures.